



URGENT FIELD SAFETY NOTICE/ DEVICE RECALL

COMMERCIAL NAME: NC Trek RX Coronary Dilatation Catheter
NC Traveler RX Coronary Dilatation Catheter
NC Tenku RX PTCA Balloon Catheter

FSCA-Identifier: March 22, 2017

Type of Action: Device Recall

Attention: Risk Manager or Healthcare Professional

Dear Valued Abbott Vascular Customer:

Abbott Vascular has initiated a voluntary field action regarding specific lots of the NC Trek RX Coronary Dilatation Catheter, NC Traveler RX Coronary Dilatation Catheter, and NC Tenku RX PTCA Balloon Catheter. Our records indicate that affected lots have been shipped to your account.

Product from the identified lots may exhibit difficulty in removing the protective balloon sheath which can result in issues with inflating or deflating the balloon. The worldwide frequency of tight sheath removal, inflation and deflation reported events is 0.12%. Potential risks associated with balloon inflation and deflation difficulties include air embolism, additional intervention, thrombosis, and myocardial infarction. In one reported case, failure to deflate the balloon necessitated surgery, leading to multiple post-operative complications and death.

This action does not affect patients having successfully undergone cardiac procedures using these devices.

How does this issue occur?:

If excessive force is required to remove the balloon sheath, the sheath may damage the device causing problems with inflating or deflating the balloon.

What action is Abbott Vascular asking you to take?:

- Please reference the attached list of affected part numbers and lot numbers
- Please immediately stop using the devices from these lots.
- Please review your inventory, complete, and return the attached Effectiveness Check Form
- Return all unused identified products to Abbott Vascular
- Share this notification with other relevant personnel in your organization

What is Abbott Vascular doing?:

Abbott Vascular has already implemented corrective actions to ensure ongoing product performance. Abbott Vascular will work with you to replace returned units with similar product, pending availability. The appropriate regulatory agencies have been notified of this action.

We regret any inconvenience this may cause you and appreciate your patience. Abbott Vascular is committed to providing high quality products and ensuring customer satisfaction. If you have any questions, please do not hesitate to contact your local Abbott Vascular Representative or Customer Service department on xxx xxxxx.

Sincerely,

[Name]
GM / Country Manager

URGENT FIELD SAFETY NOTICE / DEVICE RECALL
March 22, 2017

NC Trek RX Coronary Dilatation Catheter

Part Number	Lot Number	Part Number	Lot Number	Part Number	Lot Number
1012448-06	60414G1	1012448-15	61014G1	1012449-12	60408G1
	60721G1		61028G1		60413G1
1012448-08	60208G1		61206G1		60413G2
	60419G1		61206G2		60425G1
	60420G1	1012448-20	60310G1		60429G1
	60603G1		60421G1		60511G1
	60906G1		60421G2		60512G1
	61121G1		60527G1		60531G1
61121G2	60716G1		60607G1		
1012448-12	60225G1		1012449-06		60819G1
	60229G1	61024G2			60713G1
	60318G1	60523G1			60721G1
	60318G2	61026G1	60804G1		
	60429G1	1012449-08	60317G1		60805G1
	60429G2		60318G1		60809G1
	60607G1		60412G1		60812G1
	60705G1		60419G1		60816G1
	60804G1		60502G1		60824G1
	60818G1		60502G2		60926G1
	60921G1		60525G1		60929G1
	60921G2		60705G1	61012G1	
	61108G1		60801G1	61019G1	
	61111G1		60809G1	61019G2	
61128G1	60826G1	61020G1			
61128G2	60916G1	61115G1			
61218G1	61018G1	61121G1			
1012448-15	60202G1	1012449-12	61121G1	61122G1	
	60310G1		61121G2	61123G1	
	60401G1		60218G1	61123G2	
	60408G1		60303G1	61201G1	
	60408G2		60330G1	61208G1	
	60510G1		60330G2	61214G1	
	60616G1		60406G1	61214G2	
	60721G1		60406G2	61216G1	
	60809G1		60406G3	61216G2	
60912G1	60406G4		1012449-15	60128G1	

URGENT FIELD SAFETY NOTICE / DEVICE RECALL
March 22, 2017

NC Trek RX Coronary Dilatation Catheter (Continued)

Part Number	Lot Number	Part Number	Lot Number	Part Number	Lot Number	
1012449-15	60215G1	1012449-15	61122G1	1012450-12	60623G1	
	60215G2		61123G1		60831G1	
	60224G1		61128G1		60921G1	
	60322G1		61130G1		61104G1	
	60329G1		61202G1		61207G1	
	60330G1	60126G1	61209G1			
	60330G2	1012449-20	60223G1	1012450-15	60315G1	
	60330G3		60225G1		60317G1	
	60401G1		60226G1		60328G1	
	60407G1		60323G1		60414G1	
	60407G2		60331G1		60414G2	
	60427G1		60331G2		60613G1	
	60428G1		60516G1		60718G1	
	60428G2		60519G1		60810G1	
	60428G3		60721G1		60901G1	
	60429G1		60805G1		60922G1	
	60502G1		60824G1		61128G1	
	60603G1		60907G1		61130G1	
	60609G1		61004G1		1012450-20	60318G1
	60621G1		61116G1			60329G1
	60718G1		61128G1			60429G1
	60720G1		61130G1			60613G1
	60727G1		61206G1			60804G1
	60801G1	60405G1	60929G1			
	60809G1	1012449-25	60412G1	1012451-06	60603G1	
	60810G1		60805G1		61006G1	
	60816G1	1012450-06	60610G1	1012451-08	60323G1	
	60824G1		61101G1		60329G1	
	60906G1		60530G1		60329G2	
	60919G1	1012450-08	60803G1		60330G1	
	60926G1		61014G1		60425G1	
	60928G1		61206G1		60428G1	
61104G1	1012450-12	60212G1	60526G1			
61107G1		60412G1	60531G1			
61111G1		60414G1	60623G1			
61111G3		60528G1	60713G1			

URGENT FIELD SAFETY NOTICE / DEVICE RECALL
March 22, 2017

NC Trek RX Coronary Dilatation Catheter (Continued)

Part Number	Lot Number	Part Number	Lot Number	Part Number	Lot Number
1012451-08	60805G1	1012451-12	61108G1	1012451-15	60920G1
	60819G1		61114G1		61010G1
	60929G1		61122G1		61024G1
	61104G1		61123G1		61101G1
	61204G1		61130G1		61115G1
	61206G1		61212G1		61118G1
	61207G1		61212G2		61123G1
	61208G1		61213G1		61123G2
1012451-12	60205G1	1012451-15	60210G1	1012451-20	61202G1
	60214G1		60224G1		61204G1
	60215G1		60226G1		61214G1
	60322G1		60227G1		60128G1
	60330G1		60229G1		60307G1
	60408G1		60307G1		60310G1
	60415G2		60314G1		60318G1
	60415G3		60329G1		60329G1
	60415G4		60329G2		60419G1
	60418G1		60331G1		60419G2
	60418G2		60401G1		60519G1
	60502G1		60408G1		60603G1
	60525G2		60408G2	60824G1	
	60527G1		60505G1	60830G1	
	60527G2		60512G1	60901G1	
	60601G1		60603G1	60927G1	
	60616G1		60607G1	61115G1	
	60629G1		60628G1	61121G1	
	60720G1		60701G1	61201G1	
	60729G1		60720G1	1012451-25	
	60805G1		60727G1	60809G1	
	60818G1		60801G1	1012452-08	
	60829G1		60818G1		50114G1
	60907G1		60826G1		50316G1
	60922G1		60826G2		50319G1
	61003G1		60909G1		50513G1
	61010G1		60909G2		51022G1
	61107G1		60913G1		51217G1
		60112G1			
		60603G1			

URGENT FIELD SAFETY NOTICE / DEVICE RECALL
March 22, 2017

NC Trek RX Coronary Dilatation Catheter (Continued)

Part Number	Lot Number	Part Number	Lot Number	Part Number	Lot Number
1012452-08	60810G1	1012452-15	60718G1	1012453-12	60705G1
	60921G1		60726G1		60722G1
	61007G1		60826G1		60810G1
	61216G1		61115G1		60907G1
1012452-12	50109G1		61128G1		61004G1
	50210G1		61208G1		61012G1
	50306G1	1012452-20	50106G1	61214G1	
	50317G1		50120G1	61218G1	
	50330G1	50204G1	1012453-15	60301G1	
	50420G1	50206G1		60303G1	
	50817G1	50316G1		60329G1	
	50928G1	50603G1		60331G1	
	51023G1	51110G1		60427G1	
	51029G1	60412G1		60428G1	
	60217G1	60607G1		60613G1	
	60226G1	61018G1		60617G1	
	60308G1	1012453-08		60405G1	60721G1
	60311G1			60413G1	60819G1
	60712G1			60413G2	60822G1
	60909G1			60425G1	60909G1
	61102G1		60425G2	61026G1	
	61122G1		60621G1	61111G1	
	1012452-15		50105G1	60720G1	61121G1
			50115G1	60729G1	1012453-20
50213G1		60819G1	60226G1		
50316G1		61026G1	60505G1		
50319G1		61130G1	60623G1		
50420G1		61204G1	60810G1		
50609G1		1012453-12	60302G1	60926G1	
50929G1			60408G1	61122G1	
51103G1			60415G1		
51109G1			60502G1		
60224G1		60502G2			
60331G1		60503G1			
60405G1	60503G2				
60505G1	60621G1				

URGENT FIELD SAFETY NOTICE / DEVICE RECALL
March 22, 2017

NC Tenku RX PTCA Balloon Catheter

Part Number	Lot Number
1012448-08T	60202G1
	60301G1
	60502G1
1012448-12T	60301G1
	60401G1
	61101G1
1012448-15T	60202G1
	60401G1
	60601G1
	60801G1
	61101G1
1012449-08T	60302G1
	60401G1
	60601G1
	60801G1
1012449-12T	60202G1
	60401G1
	60502G1
	60601G1
	60701G1
	60902G1
	61004G1
	61101G1
1012449-15T	60202G1
	60301G1
	60401G1
	60502G1
	60601G1
	60701G1
	60801G1
	60902G1
	61101G1
1012450-08T	60401G1

Part Number	Lot Number
1012450-08T	60601G1
	60801G1
1012450-12T	60301G1
	60701G1
1012450-15T	60301G1
	60502G1
	60701G1
1012451-08T	60301G1
	60502G1
	60701G1
	60902G1
1012451-12T	60301G1
	60502G1
	60701G1
	61004G1
1012451-15T	60301G1
	60401G1
	60601G1
	60701G1
	60805G1
	61004G1
1012452-08T	50204G1
	50406G1
1012452-12T	50202G1
	50901G1
1012452-15T	50204G1
1012453-08T	60202G1
	60502G1
	60701G1
1012453-12T	61101G1
	60202G1
	60506G1

NC Traveler RX Coronary Dilatation Catheter

Part Number	Lot Number
1013152-12	61209G1
	61209G2
1013152-15	61102G1
1013152-20	61102G1
1013153-08	61012G1
	61209G1
1013153-12	61101G1
1013153-15	60415G1
	61209G1
1013153-20	61102G1
1013154-08	61101G1
1013154-12	61101G1
1013154-15	61102G1
1013154-20	60415G1
1013155-08	61102G1
1013155-12	61101G1
1013155-15	60415G1
	61209G1
1013155-20	61102G1
1013156-12	50415G1
	50919G1
	51110G1
1013156-15	61209G1
	50919G1
	51210G1
1013157-08	61101G1
	61102G1
1013157-12	60415G1
	61102G1
1013157-15	60415G1
	61209G1

URGENT FIELD SAFETY NOTICE/ DEVICE RECALL

COMMERCIAL NAME: NC Trek RX Coronary Dilatation Catheter
NC Traveler RX Coronary Dilatation Catheter
NC Tenku RX PTCA Balloon Catheter

FSCA-Identifier: March 22, 2017
Type of Action: Device Recall

Effectiveness Check Form

Customer Account # _____
Account Name _____
Address _____
(Information required for regulatory effectiveness check)

After reviewing your inventory of NC Trek RX Coronary Dilatation Catheter, NC Traveler RX Coronary Dilatation Catheter, and NC Tenku RX PTCA Balloon Catheter, please check one box in the section below. If affected inventory was identified, please contact Customer Services to obtain a Returned Goods Authorization (RGA) number. After signing this form, please return the form and any identified products to Abbott Vascular.

Form with two checkboxes:
1. A thorough search for all affected products has been completed and no affected units remain in inventory. No devices will be returned.
2. Affected NC Trek RX Coronary Dilatation Catheter, NC Traveler RX Coronary Dilatation Catheter, and or NC Tenku RX PTCA Balloon Catheter devices have been identified and are being returned.
RGA Number: _____

Customer Name/ Title (print) _____ Signature _____ Date _____

- This form is to be returned to Abbott Vascular
- If returning product, call Abbott Vascular Customer Service XXX XXX XXXX to receive RGA number. Record RGA number above.
- Fax this completed form to XXX XXX XXXX or scan an e-mail to XXXXXXXXXXXX
- Return a copy of this completed form with the returned product.

